

CREATE BETTER PATIENT OUTCOMES

THE ART OF
OXYGEN

FreeStyle™

Portable Oxygen Concentrator

FreeStyle™ — For Oxygen Patients with No Boundaries!

The FreeStyle portable oxygen concentrator (POC) makes its own oxygen and is the most compact and lightweight, up to a 3-setting. FreeStyle offers a uniquely efficient flexible pulse-flow delivery system.


FreeStyle is approved for airline travel and allows active oxygen patients to go wherever they desire. Patients and providers are attracted to its sleek design and capabilities.

The internal battery enables up to 3½ hours of use between recharges and when more time is needed away from an available power source for extended activity – FreeStyle's optional AirBelt™ is there to provide hours of extended battery duration. AirBelt connects easily to the unit and depending on the setting, there can be up to ten hours of ambulation between recharges.

With this unit, oxygen users can now participate in an all-day, outdoor music festival, a sporting event, a continuing education class, or even enjoy the added benefits of FreeStyle during extended travel.

FEATURES

- △ Ultra-lightweight at 4.4 lbs (2.0 kg)
- △ Operates on AC/DC and rechargeable internal battery power
- △ Exceptional battery life
- △ FAA-approved for on-board, in-flight use
- △ Fashionable carrying case with adjustable shoulder strap
- △ Optional harness available to be worn as a backpack
- △ Up to 10 hours between battery recharges with the optional AirBelt at setting 1

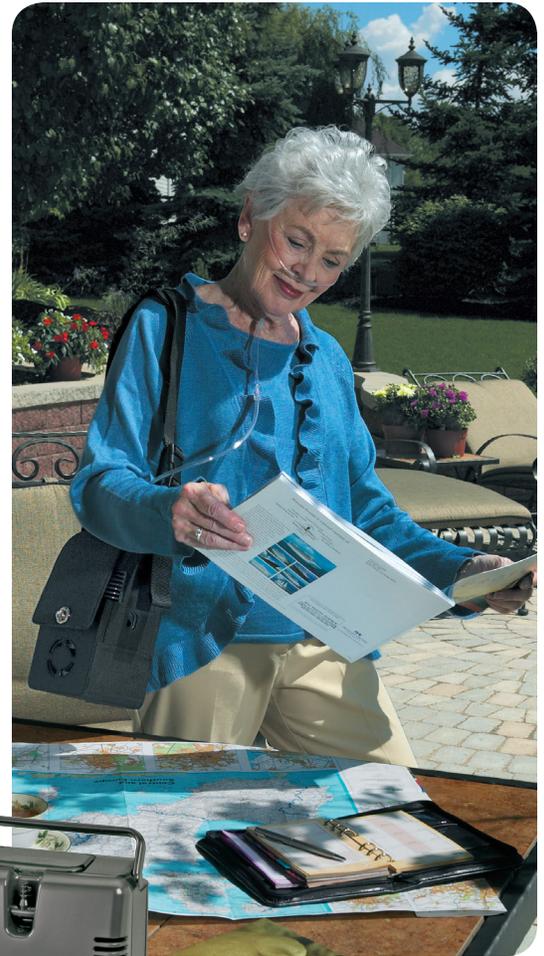

Call 1.800.482.2473 or 1.800.874.0202 to order!


CAIRE®
A Chart Industries Company
AirSep® SeQual®
www.CAIREmedical.com



Accessories and Controls

FreeStyle is a wearable POC that can be worn over the shoulder in its soft-sided carrying-bag. It can also be easily converted to accommodate an optional harness to be worn as a backpack. The convertible bag even allows patients to wear the unit at the waist with any standard belt or the optional AirBelt.

The carry-all-bag accessory is able to hold all power accessories for the unit and even extra AirBelt's, if needed.

To maximize ambulation time, FreeStyle can be used as an ensemble consisting of the portable oxygen concentrator, comfortable carrying case, and matching AirBelt. The flexible, swivel oxygen outlet allows for comfortable positioning of the cannula. Through the transparent control panel cover, the unit also displays a battery test button, which is used to verify the status of the internal battery. Soft-touch buttons, enhanced with Braille on the unit to note the pulse settings, further hallmark this product's exceptional design and attention to detail.

(Not shown) FreeStyle is equipped with a universal AC/DC power supply that can be used to operate the unit worldwide and simultaneously recharge the internal battery. Various power cord styles available.





Convertible Carrying Bag (soft-sided with comfort strap) Part No. MI283-1



Carry-All-Bag for FreeStyle Accessories Part No: MI320-1



Backpack Harness for Convertible Carrying Bag Part No: MI284-1

Product Specifications

Setting	1-3 pulse settings
Oxygen Concentration*	90% oxygen +5.5/-3%
Dimensions	8.6 in. H x 6.1 in. W x 3.6 in. D (21.8 cm H x 15.5 cm W x 9.1 cm D)
Weight	4.4 lb (2 kg); 1.8 lb (0.8 kg) — optional AirBelt
Power	Universal Power Supply; AC Power: 100–240 VAC, 50/60 Hz (1.0 amp at 120 VAC) DC Power: 11–16 VDC (6.0 amps at 12 VDC) FreeStyle Unit: 28 W nominal at 3 LPM
Battery Duration	3 setting - 2 hours 2 setting - 2.5 hours 1 setting - 3.5 hours
Recharge Time	3.5 hours
(Optional) AirBelt Duration	3 setting - An additional 3 hours; 5 hours total with AirBelt 2 setting - An additional 3.5 hours; 6 hours total with AirBelt 1 setting - An additional 6.5 hours; 10 hours total with AirBelt
Recharge Time	3 hours
Battery Cycle Life	Approximately 300 cycles, then 80% capacity or below
Warm-up Time	2 minutes
Audible Alarms and Visual Indicators	Start-up, Pulse flow, Battery condition/level, Cannula disconnect, System overdraw, General malfunction
Temperature Range	Operational Temperature: 41°F to 104°F (5°C to 40°C); Storage Temperature: -4°F to 140°F (-20°C to 60°C)
Sound Level	38 dBA at 1 setting 41 dBA at 2 setting 44 dBA at 3 setting
Altitude	Up to 12,000 ft (3,657.6 m) (Higher altitudes may affect performance.)

*Based on an atmospheric pressure of 14.7 psia (101 kPa) at 70° F (21 °C). Operating unit outside of operational temperature range can affect performance.

FreeStyle™

Portable Oxygen Concentrator



WARRANTY

FreeStyle carries a three-year parts warranty. The internal battery, as well as AirBelt, is covered with a one-year warranty.

INTERNATIONAL APPROVALS

CE 0459

ISO 9001:2008 and ISO 13485:2003-certified.

AIRLINE APPROVALS

FreeStyle is approved by the Federal Aviation Administration (FAA) for onboard, in-flight use.

Ordering Information

Description

FreeStyle Portable Oxygen Concentrator with US-style power cord

Catalog Number*

AS095-1

FreeStyle Portable Oxygen Concentrator with European-style power cord

AS095-2

FreeStyle Portable Oxygen Concentrator with Australian-style power cord

AS095-3

FreeStyle Portable Oxygen Concentrator with UK-style power cord

AS095-4

Accessories

AirBelt with battery, AC power supply, and US-style power cord

BT017-1

AirBelt with battery, AC power supply, and European-style power cord

BT017-2

AirBelt with battery, AC power supply, and Australian-style power cord

BT017-3

AirBelt with battery, AC power supply, and UK-style power cord

BT017-4

*Catalog Numbers reflect US English Part Numbers, for language-specific part numbers please contact Customer Service.

CAIRE, Inc.
2200 Airport Industrial Drive, Suite 500
Ball Ground, GA 30107
Ph 716-691-0202 • Toll-Free 1-800-482-2473 (CAIRE) or 1-800-874-0202 (AirSep)
Fax 888-932-2473 (CAIRE) or 716-691-4141 (AirSep)

Chart BioMedical, Ltd.
Unit 6, Ashville Way
Wokingham RG41 2PL, United Kingdom
Ph +44(0) 1189 367060 • Fax +44(0) 1189 799245